
clinicus I, clinicus II
K I, K II
roid I, roid II
manuell
solid
varius
minimus
oscillophon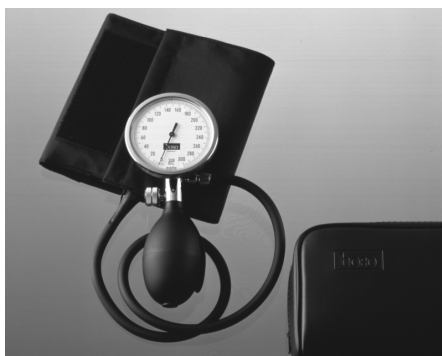

- Ⓓ Gebrauchsanweisung
- Ⓕ Mode d'emploi
- Ⓘ Manuale di istruzioni
- ⒼⒽ User Instructions
- Ⓔ Manual del Usuario

 **BOSCH + SOHN GmbH u. Co. KG**
 Bahnhofstraße 64 · 72417 Jungingen · Germany
 Telephone: +49 (0) 74 77 - 92 75-0 · Fax: +49 (0) 74 77 - 10 21
 Internet: www.boso.de · e-Mail: zentrale@boso.de

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CE 0124

Contents of the kit:

- 1 blood pressure instrument with cuff
- 1 set of instructions for use
- 1 case (optional)

Preliminary remarks:

This blood pressure instrument complies with the European regulations that are the basis for the German Medicinal Products Act (CE marking) and the international standard ISO 81060-1.

All Calibration testing – at least every two years –, has to be carried out either by the manufacturers or the local authorised dealer in accordance with the relevant distributor directives.

Purpose:

Non-invasive recording of systolic and diastolic blood pressure in humans. The instrument can be used with any arm circumference as shown on the corresponding cuff.

Important:

Keep this unit from being knocked or dropped, and protect it from dirt and moisture. Care has to be taken not to puncture the cuff with any sharp instruments (Scissors, Needles).

Do not inflate to above 300 mmHg !

Measurement time should be ca. 2 minutes maximum.

Between 2 measurements allow an interval of at least 2 minutes.

Please use only boso cuffs.

The patient must follow these basic rules when having his/her blood pressure measured:

- sit comfortably
- not cross his/her legs
- rest his/her back and arms on the chair or another surface
- have the centre of the cuff placed on the upper arm at the level of the heart

- relax as much as possible and keep silent while his/her blood pressure is being measured
- be calm for 5 minutes before the first reading

The following phases are recommended in the case of auscultatory measurement

- adults: phase V (K5)
- children aged 3 to 12: phase IV (K4)
- pregnant women: phase V (K5), except when sounds are audible when the cuff is deflated: in this case phase IV (K4) Korotkoff sounds should be used.

K5 is the point at which tones heard through the stethoscope are no longer audible

K4 is the point at which tones heard through the stethoscope change from a clear sound to a muffled sound.

Inscriptions:

“mmHg” is the unit in which blood pressure is measured

“shock protected” = the instrument has been tested and found to work correctly after being dropped from a height of up to one metre.

Fitting the Cuff:

The cuff must not be fitted over wounds as this could lead to further injury.

Blood pressure must always be measured on the arm with the higher pressure values.

To do this, first measure blood pressure on both arms and subsequently always on the arm which produced the higher reading.

Place cuff in such a manner that the lower cuff edge finishes ca. 2 – 3 cm above the elbow and the centre of the rubber bag rests on top of the artery.

Taking the Blood Pressure:

Inflate the cuff to above the systolic pressure. The air release speed can be adjusted by rotating the valve screw.

The World Health Organisation (WHO) recommends a deflation speed of 2 – 3 mmHg per second.

After the measurement is completed, fully open the air release valve to allow for the fast deflation of the cuff.

Technical Data:

Measurement range: 0 – 300 mmHg

Accuracy of Pressure indication: ± 3 mmHg

Storing conditions:

$-30^{\circ}\text{C} - 70^{\circ}\text{C}$, 15 – 85% rel. humidity

Operating conditions:

$10^{\circ}\text{C} - 40^{\circ}\text{C}$, 15 – 85% rel. humidity

Cleaning of Unit and Cuff:

- To clean the unit only use a dry, soft cloth.
- To clean the cuff – remove the rubber bag. The exterior of the cuff can then be washed by hand at a maximum temperature of 30 degrees Celsius.

Disinfection:

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend the disinfectant mikro-zid sensitiv liquid (Schülke & Mayr).

To disinfect the velcro of the cuff, we recommend spray disinfection.

Warranty Conditions:

We give 2 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty. No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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